

AUDISOL[®] EAR WAX REMOVER



**READ THIS LEAFLET
CAREFULLY BEFORE USE.
USE ONLY AS DIRECTED.
IF SYMPTOMS PERSIST
CONSULT A HEALTHCARE
PROFESSIONAL.**

What causes ear wax accumulation?

Ear wax or cerumen is a normal secretion of the glands in the ear. However excessive ear wax may cause a number of unwanted effects including hearing loss, discomfort, pain and coughing.

Hearing loss occurs when accumulated wax occludes (blocks) the ear canal, reducing or preventing sound waves from easily reaching the eardrum.

Discomfort or pain may be evident in cases where compressed wax pressures the ear drum.

Coughing may be occurring in cases where the impacted wax stimulates nerves shared both by the ear canal and throat, provoking a tickle in the throat which may lead to cough.

How does Audisol[®] Ear Wax Remover work?

Audisol[®] is an effective natural spray intended to dissolve and wash away accumulated ear wax. Audisol[®] combines a mechanical wash compound with the safety and efficacy of natural olive oil. Using Audisol[®], earwax is dissolved and gradually removed from the ear by using a natural ear cleaning mechanism.

The Audisol[®] spray mechanism ensures perfect dispersion of the solution in the ear canal.

Composition

Mineral Oil, Phytosqualan, Spearmint Oil.

When to use Audisol[®] Ear Wax Remover?

Audisol[®] is indicated for the dissolution and removal of ear wax in the external ear canal for both children 2 years+ and adults.

Warnings

- Audisol® ear wax remover should not be used in patients with perforated eardrum, inflamed ears, using grommets, had recent ear surgery or suffering from ear infections.
- Do not spray into the eye. In case of accidental contact with eye, wash thoroughly with running water
- In case of any irritation or sensitivity discontinue use immediately and get advice from a medical professional.
- **FOR EXTERNAL USE ONLY**

How to use Audisol® Ear Wax Remover?

Spray 1-2 puffs in each ear 3 times a day until the wax is dissolved and symptoms are resolved.

In difficult cases, straighten the ear canal by pulling the upper ear lobe slightly upward and toward the back of your head. This maneuver allows penetration of the active ingredients to the inner parts of the ear canal.

Storage Conditions:

- Store in a cool place. Do not refrigerate.
- Do not expose to temperature over 50°C.
- Keep out of sight and reach of children

DISTRIBUTED BY

UK	Noru Pharma Ltd, Medicity - D6, Thane Road, Nottingham NG90 6BH
AUSTRALIA	Noru Pharma, Unit 10, 50 Victoria Road, Drummoyne, NSW 2047
NEW ZEALAND	Noru Pharma Ltd, 48B Arundel Street Hillsborough Auckland 1041
HONG KONG	Noru Pharma (H.K.) Ltd, Flat U, 7/F, Valiant Industrial Centre, 2-12 Au Pui Wan Street, Fotan, NT
SINGAPORE	Noru Pharma Pte. Ltd, 29 Jalan Demak, Singapore 488975

MEANING OF SYMBOLS USED



CE marking with the four digit identification number of the notified body involved in the conformity assessment procedure. The CE marking is the manufacturer's declaration that the product meets the requirements of the applicable European Directive.



Symbol indicating that the product packaging is able to be recycled.



Manufacturer information.



The product should not be used after the expiry date.



Batch number of the product.

A product of



Noru Pharma
Live Better

More Information: norupharma.com



Manufactured by:
Danima Smart Solutions B.V
Herengracht 74
1015 BR Amsterdam
The Netherlands

Audisol® is a registered trademark.